ÓRTESE CORRECTIVA DUPLA DO HALLUX VALGUS

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DOUBLE PROTECTION HALLUX VALGUS CORRECTIVE ORTHOSIS

INDICATIONS

To correct and limit the progression of hallux valgus while relieving joint and plantar pain.

CONTRA-INDICATIONS

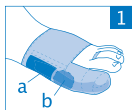
- Big toe that does not move sideways.
- Do not use on unprotected wound.
- Persons suffering from arthritis, diabetes, polyneuritis, lower limb neuropathy, varicose ulcers or severe oedema: never use the product on a wound, monitor the condition of your foot during use, and in all cases seek advice from your doctor or chiropodist.

PRECAUTIONS FOR USE

- This device should not be used when the big toe does not have sideways movement.
- Do not use this product during the night.
- There are two versions: left foot and right foot. The black part of the device should be in contact with the insole.
- Wash before first use.

- PLACEMENT:

- Insert the forefoot in the compression band.
- Cover the big toe with the hood.
- Position the elastic loop on the 2nd or 3rd toe (depending on the alignment of the plantar protective device).
- Place the Epithelium™ patch (a) behind the bunion (b). (See Diagram 1)

- ADJUSTMENT:

- On the first day, do not try to straighten the big toe.
- Over 10 days, progressively straighten the big toe by pulling the device backwards. (See Diagram 2)

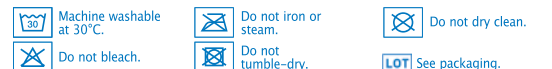
THE DEVICE SHOULD BE WORN DAILY FOR MAXIMUM EFFICIENCY.

CLEANING

- Machine-wash at 30°C in the washing net provided or wash by hand (without rubbing).
- Leave to air-dry. Do not iron.

COMPOSITION: 56% polyamide/elastane fabric, 44% silicone.

The materials used have passed all the regulatory tests required by ISO standard 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation. MILLET Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2012.

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